



PROTEOMIS

PROTEOMIS
CEIA LAB, sprl
Bd Saint-Michel 119
1040 BRUXELLES - Belgique
Tél: +32 2 736 04 58
Fax: +32 2 736 58 02

Request form

Profile nr. :	Reserved
	Entry date :

Patient information*

Last name : _____

First name : _____

Birthdate : (dd/mm/yyyy) : _____ Gender : M F

Address : _____

Zip code : _____ City : _____

Country : _____

Tel : _____ Fax: _____

E-mail : _____

Referring physician (name or stamp)*

Sample information *

Date of collection : _____

Hour : _____

* Please, complete in capital letters

Clinical information

Weight : _____ Height : _____

Antecedents (give date) : _____

Subjective complaints : _____

Known diagnosis : _____

Suspected diagnosis : _____

Objective signs : _____

Important treatment followed by the patient : _____
If possible, use ICD-10 codes from the WHO International Classification of Diseases

Requested analyses €

PROTEOMIS Profile	<input type="checkbox"/>	136,00
⇒ I dry tube (no anticoagulant), 7.5 ml, gold-top ● (or red)		
Personalized interpretation and report	<input type="checkbox"/>	50,00
Fedex prepaid shipment European Union (optional)	<input type="checkbox"/>	35,00
Fedex prepaid shipment Rest of the world (optional)	<input type="checkbox"/>	80,00
Total amount		_____

Payment method

Bank transfer :
IBAN : BE05 3101 1277 1175
BIC : BBRUBEBB

Credit card

EUROCARD/MASTERCARD VISA

Name : _____

Card number : _____

Control Code : _____ Expires : _____

Date : _____ Signature : _____

Patient consent

I understand that the analyses mentioned above are not reimbursed by Social Security and that I shall be charge for the cost of these.

Date : _____ Signature : _____

FO-TECH-046 - Request form - EN - 6 - 01/08/2022

INSTRUCTION FOR BLOOD COLLECTOR

*** PROTEOMIS profile ***

Tubes to collect	PROTEOMIS Profile (CEIA Method)	1 dry tube (no anticoagulant), 7.5 ml, gold-top ● (or red)
------------------	---------------------------------	--

Provided equipment

- Tubes :
 - 1 x 7,5 ml Vacutainer SST II Advance tube(s) with gold ● closure, gel and clotting activator.
 - 1 Vacutainer blood collection needle to be connected to the Vacutainer tube using the Vacutainer holder.
- A waterproof bag
- A protective container
- This analysis request form
- A Fedex prepaid request form to be completed
- A user's guide for blood collection (if Sartedt tubes)
- A user's guide for shipment
- A padded envelope, UN3373 marked for postal shipment
- A Fedex « Clinical Pack » bag (if shipped via Fedex)

Method for collecting and sending

Sample :

- When possible, the blood sample is taken on an empty stomach.
- **Write on the tube : patient's first and last name, birthdate, collection date**
- If possible, centrifuge the dry tubes only at 1800-2200 g (=ca 3000 rpm) during 10 minutes and collect the serum.
- Place the tube in the protective container
- Then, place the container into the waterproof bag, pull away the blue release liner and press to close the bag
- Place the prescription, the analysis request form in the side pocket of the bag
- Place in the padded envelope
- Ship to :
**PROTEOMIS
CEIA LAB sprl
Bd Saint-Michel 119
B-1040 Brussels - Belgium**
- Important :
 - ⇒ do not freeze the samples
 - ⇒ if shipment is delayed, keep the samples in the refrigerator at + 4°C.
 - ⇒ do not forget to fill in the analysis request form and to give full details of the patient
- **If shipment by Fedex via Proteomis :**
 - ⇒ Contact us to confirm the shipping prices
 - ⇒ Contact us to obtain a Fedex shipping note
 - ⇒ Place the envelope with the sample and request form in the Fedex Clinical Pack bag
 - ⇒ Contact your Fedex local customer services (France : **0820 123 800**, 08456 07 08 09) to schedule a pickup with Fedex or plan it with us
 - ⇒ Give the pack to the driver
 - ⇒ **You do not have to pay anything at pickup time PROTEOMIS-CEIA LAB has an account at Fedex Contact us if necessary**